

## **NeverTouch Gold-Tip Fiber**

THE GOLD STANDARD IN ENDOVENOUS LASER TREATMENT





MEDICAL  
SPECIALTIES  
AUSTRALASIA



**angiodynamics**

## The NeverTouch\* Gold-Tip Fiber for Less Pain and Bruising


The new NeverTouch gold-tip laser fiber establishes a new standard for patient comfort, visibility and ease of use. The gold-tip fiber eliminates laser tip contact with the vein wall, which in turn minimizes perforations of the vein wall that typically result in pain and bruising as compared to traditional bare-tip fibers. The gold-tip also maximizes ultrasonic visibility, making it easier for physicians to use.

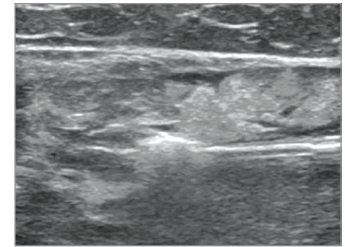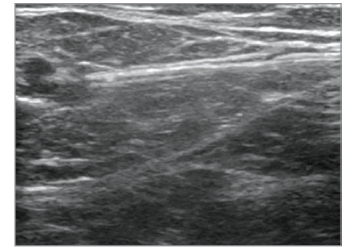

Echogenic


## The Proprietary Science Behind NeverTouch Fibers

The innovation behind the NeverTouch fiber is a glass weld at the distal tip of a 600 um fiber. The weld results in an effective fiber diameter of 905 um and lowers the actual power density by 56 percent from that of a standard bare tip 600 um fiber.

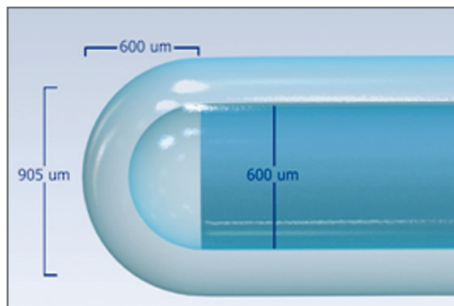


The net effect is a homogeneous ablation with less focal charring of the vein wall than that which is seen with bare tip fibers. AngioDynamics' patent-pending technology is different from competitors with standard covered-tip fibers that have the same power density as a 600 um fiber.

## The NeverTouch Fiber Can Successfully Treat Larger Veins

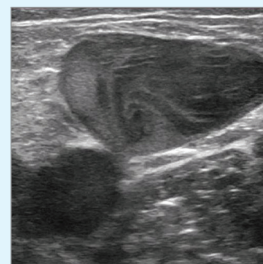

BEFORE


39.2 mm GSV before

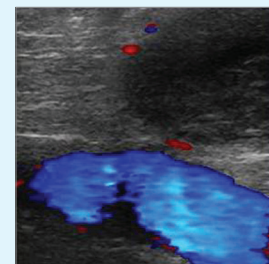

AFTER

Vein after Venacure EVLT\* laser procedure with NeverTouch fiber

# PROCEDURE KIT FEATURES AND BENEFITS

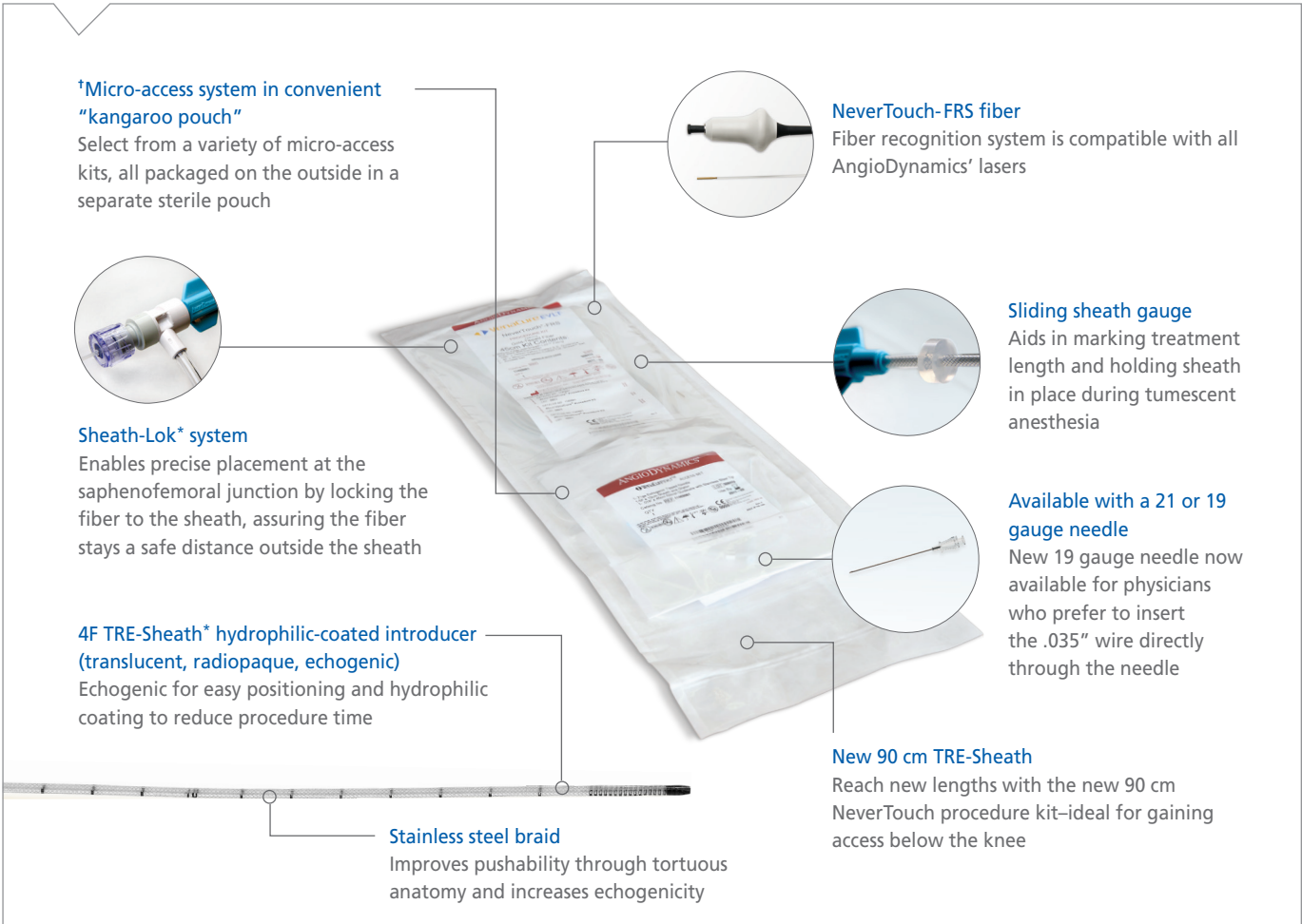

## \*Micro-Access System

AngioDynamics offers several micro-access systems to satisfy a variety of clinical needs. The chart below specifies the kit options and components that accompany the NeverTouch gold-tip fiber.

### New NeverTouch Fiber .018" System

The new .018" system eliminates the need for a 5F micro-access sheath and eliminates the need to exchange wires and size up, resulting in fewer procedure steps, and a faster procedure time.

	21 G Needle	19 G Needle	.018" Micro-access guidewire	Micro-introducer sheath with dilator	.035" long-access double-ended guidewire	.018" long-access guidewire	TRE-Sheath with dilator
Standard System	X		X	X	X		X
.018" System	X					X	X
19 G Micro-access System		X			X		X

## NEVERTOUCH GOLD-TIP FIBER PROCEDURE KITS

Description	Part #	Kit Length	Long-Access Guidewire	Needle
NeverTouch Kit	11402001	45 cm	.035"	21 gauge
NeverTouch Kit	11402002	65 cm	.035"	21 gauge
NeverTouch-FRS Kit	11403001	45 cm	.035"	21 gauge
NeverTouch-FRS Kit	11403002	65 cm	.035"	21 gauge
NeverTouch-FRS Kit	11403004	90 cm	.035"	21 gauge
NeverTouch-FRS Kit	11403005	45 cm	.035"	19 gauge
NeverTouch-FRS Kit	11403006	65 cm	.035"	19 gauge
NeverTouch-FRS Kit	11403007	90 cm	.035"	19 gauge <b>NEW</b>
NeverTouch-FRS Kit	11403003	25 cm	.018"	21 gauge
NeverTouch-FRS Kit	11403012	45 cm	.018"	21 gauge
NeverTouch-FRS Kit	11403013	65 cm	.018"	21 gauge

AngioDynamics' VenaCure EVLT endovenous laser vein treatment offers patients a proven, minimally-invasive choice for treating the source of their varicose veins and provides them with immediate recovery and a return to normal daily routines. The VenaCure EVLT system includes 1470 nm and 810 nm lasers, a choice of procedure kits, procedure accessories, marketing materials and support, and more. For more information on these products, including our proprietary NeverTouch fiber technology, please visit [www.VenaCure-EVLT.com](http://www.VenaCure-EVLT.com).

### IMPORTANT RISK INFORMATION

**INDICATION FOR USE:** The AngioDynamics, Inc. VenaCure EVLT NeverTouch Procedure Kits are indicated for endovascular coagulation of the Great Saphenous Vein (GSV) in patients with superficial vein reflux, for the treatment of varicose veins and varicosities associated with superficial reflux of the Great Saphenous Vein (GSV), and for the treatment of incompetence and reflux of superficial veins of the lower extremity. This product should be used only with lasers cleared for use in the treatment of varicose veins, varicosities with superficial reflux of the GSV, and in the treatment of incompetent refluxing veins in the superficial venous system in the lower limbs.

**CAUTION:** Federal (USA) law restricts the sale of this device by or on the order of a physician.

**CONTRAINDICATIONS:** Patients with thrombus in the vein segment to be treated, patients with an aneurysmal section in the vein segment to be treated or patients with peripheral artery disease as determined by the Ankle Brachial Pressure Index with a value of <0.9 should not have their varicosities ablated.

**WARNINGS AND PRECAUTIONS:** Treatment of a vein located close to the skin surface may result in skin burn. Tissue not targeted for treatment must be protected from injury by direct and reflected laser energy. All persons in the treatment room **MUST** wear protective glasses with the proper rating for the wavelength being used.

**CAUTION:** This device is ethylene oxide sterilized and intended for single patient use only. Do not reuse or

resterilize the fibers. Contents sterile in unopened, undamaged package. Do not use if opened or any sign of product damage is visible. Carefully read all directions and observe all Warnings and Precautions prior to performing the procedure.

**POTENTIAL COMPLICATIONS:** Adverse reactions may include, but are not limited to: vessel perforation, thrombosis, pulmonary embolism, phlebitis, hematoma, infection, skin pigmentation alteration, neovascularization, paresthesia due to thermal damage of adjacent sensory nerves, anesthetic tumescence, non-target irradiation, vasospasm, hemorrhage, necrosis, skin burns and pain.

Indications, contraindications, warnings and instructions for use can be found in the instructions for use supplied with each device. Observe all instructions prior to use. Failure to do so may result in patient complications.



USA > 14 Plaza Drive, Latham, NY 12110 > tel: 800-772-6446 or 518-798-1215 > fax: 518-798-1360  
International > Haaksbergweg 75 (Margrietoren), 1101 BR, Amsterdam Z-O > The Netherlands  
tel: +31 (0)20 753 2949 > fax: +31 (0)20 753 2939

[www.angiodynamics.com](http://www.angiodynamics.com) | [www.VenaCure-EVLT.com](http://www.VenaCure-EVLT.com)

\*AngioDynamics, the AngioDynamics logo, VenaCure EVLT, the VenaCure EVLT logo, NeverTouch, TRE-Sheath and Sheath-Lok are trademarks and/or registered trademarks of AngioDynamics, Inc., an affiliate or subsidiary. May be covered by or for use under one or more of the following U.S. patents: 5,714,360; 6,986,766; 7,033,347; 7,559,329; 7,483,457; 7,907,643; other U.S. patents and foreign counterparts pending.

ADC1042-08/2015

**MEDICAL  
SPECIALTIES  
AUSTRALASIA**

54 Gibbes St, Chatswood 2067 NSW, Australia  
PO Box 764, Willoughby 2068 NSW, Australia

✉ [custserv@msa.com.au](mailto:custserv@msa.com.au) ☎ 02 9417 7955  
🌐 [www.msa.com.au](http://www.msa.com.au) 📠 02 9417 5779

© 2012 AngioDynamics, Inc. MLC 201 US Rev L 09/12