


A Low-Profile Design. A High Level of Patient Comfort.

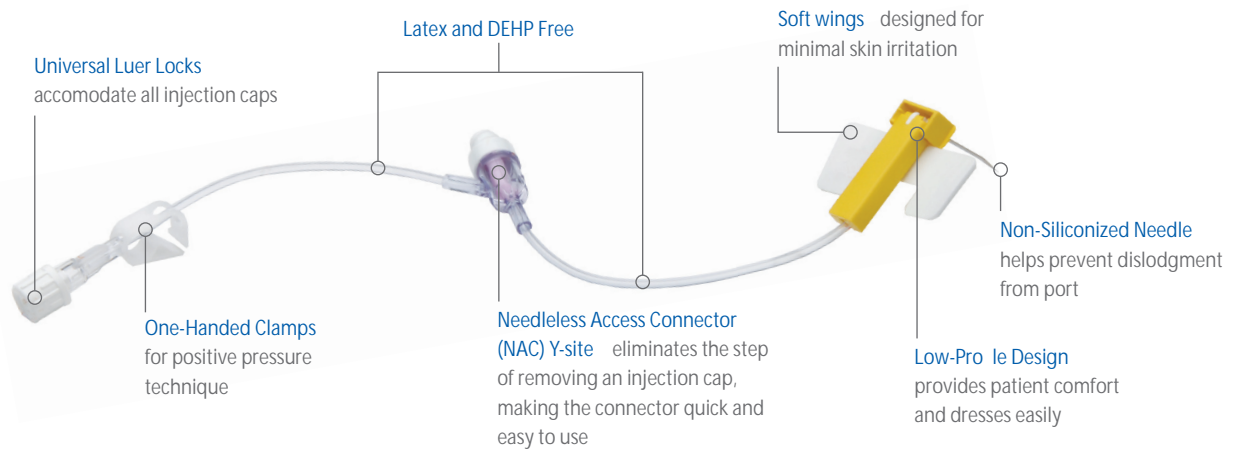

Safe Has Never Been This Easy



Raise needle cover ninety degrees.



Hold base firmly.



Lift wings straight up.



>> Select models Power-injectable with the Smart Port (refer to product list for codes)

CT

Description	UPN	Power Injectable
19 Ga. x .75"	<input type="checkbox"/> H787LG19755	Yes
19 Ga. x 1"	<input type="checkbox"/> H787LG191005	Yes
20 Ga. x .50"	<input type="checkbox"/> H787LG20505	Yes
20 Ga. x .75"	<input type="checkbox"/> H787LG20755	Yes
20 Ga. x 1"	<input type="checkbox"/> H787LG201005	Yes
20 Ga. x 1.5"	<input type="checkbox"/> H787LG201505	Yes
22 Ga. x .50"	<input type="checkbox"/> H787LG22505	No
22 Ga. x .75"	<input type="checkbox"/> H787LG22755	No
22 Ga. x 1"	<input type="checkbox"/> H787LG221005	No

Description	UPN	Power Injectable
19 Ga. x .75" w/ NAC Y-site	<input type="checkbox"/> H787LG1975NY5	No
19 Ga. x 1" w/ NAC Y-site	<input type="checkbox"/> H787LG19100NY5	No
20 Ga. x .50" w/ NAC Y-site	<input type="checkbox"/> H787LG2050NY5	No
20 Ga. x .75" w/ NAC Y-site	<input type="checkbox"/> H787LG2075NY5	No
20 Ga. x 1" w/ NAC Y-site	<input type="checkbox"/> H787LG20100NY5	No
20 Ga. x 1.5" w/ NAC Y-site	<input type="checkbox"/> H787LG20150NY5	No
22 Ga. x .50" w/ NAC Y-site	<input type="checkbox"/> H787LG2250NY5	No
22 Ga. x .75" w/ NAC Y-site	<input type="checkbox"/> H787LG2275NY5	No
22 Ga. x 1" w/ NAC Y-site	<input type="checkbox"/> H787LG22100NY5	No

DISCONTINUED

PHYSICIAN'S SIGNATURE

IMPORTANT RISK INFORMATION

INDICATION FOR USE: The AngioDynamics LifeGuard Safety Infusion Set is designed to access implanted vascular ports to administer fluids and/or to withdraw blood. The LifeGuard Safety Infusion Set is a 90° non-coring needle composed of wings, tubing, a universal Luer connector, optional Needleless Y-site, tubing clamp(s), needlestick guard and needle trap that facilitates the safe removal of the needle by encapsulating the needle during vascular port de-accessing (to help prevent needlestick injuries).

CONTRAINDICATIONS: Do not use with patients showing signs or symptoms of infection at the insertion site or who have exhibited prior intolerance to the materials of construction.

WARNINGS AND PRECAUTIONS: Review all fluids to be used with this product for any incompatibility with plastics and plastic adhesives. A 10 ml syringe or larger is recommended. Clamp infusion set when not in use. Device is intended for single use only. It is non-pyrogenic

and non-toxic. Inner contents are sterile unless package is opened or damaged. Do not resterilize. Carefully read and follow all instructions prior to use of the device. Failure to do so may result in patient complications.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.



USA > 14 Plaza Drive, Latham, NY 12110 > tel: 800-772-6446 or 518-798-1215 > fax: 518-798-1360
 International > Haaksbergweg 75 (Margrietoren), 1101 BR, Amsterdam Z-O > The Netherlands
 tel: +31 (0)20 753 2949 > fax: +31 (0)20 753 2939

www.angiodynamics.com

*AngioDynamics, the AngioDynamics logo, Smart Port and LifeGuard are trademarks and/or registered trademarks of AngioDynamics.

Inc., an affiliate or a subsidiary.

© 2014 AngioDynamics, Inc. MLC 391 US Rev C